

**OPERATOR PREVENTIVE MAINTENANCE CHECKLIST
PROCESSING UNIT, RADIOGRAPHIC FILM, AUTOMATIC**

For use of this form, see USMEPCOM Reg 750-1

Manufacturer

Model

Serial Number

CAUTION: Fixer will contaminate developer. When removing or installing fixer rack, use the rack drip tray and splash guard between the developer and fixer tanks.

	FREQUENCY
Clean unit - remove rust, dirt, lint, chemical and other deposits (remove all 4 access panels).	Daily
Remove the crossover and squeegee assemblies and clean rollers with warm water and damp cloth; wipe dry.	Daily
Wipe off all chemical deposits in the processing section.	Daily
Remove all racks, rinse with running water, and wipe with clean cloth.	Weekly
Check solution tanks for foreign matter and solution clarity.	Weekly
Check ladder chain tension to ensure racks turn freely and rollers do not skip.	Weekly
Check turnaround assembly and space between turnaround side plates for equal space on both sides.	Weekly
Check solution recirculation pumps for operation.	Weekly
Clean replenishment strainers.	Weekly
Check dryer's air-tube slits for cleanliness.	Weekly
Using an approved system cleaner, (reference manufacturer's literature) clean the developer tank.	Quarterly
Change developer filter cartridge and clean air filter.	Quarterly
Clean replenisher and processor tanks.	Quarterly

NOTE: All PM is to be performed as specified in the manufacturer's instructions found in the equipment operator's guide/manual or, in its absence, maintenance monitors will separately furnish written instructions.

